


CardioTens

ambulatory blood pressure
and ECG monitor

**an innovative ambulatory
diagnostic system**
to analyze the complex
blood-pressure related car-
diovascular risk profile

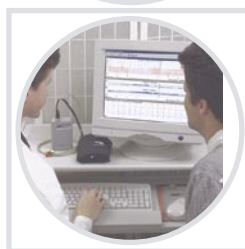

24-hour ABP profile with
sequential recording of ECG strips

24-hour ST and heart rate profile

continuous real-time ECG analysis


automatic event detection with extra
BP and ECG recordings during ST
abnormalities, heart rate variations or
syncopal episodes that exceed pro-
grammable pre-set levels

flexible programming with real-time
ECG monitoring for better electrode
placement and perfectly adjusted
event detection triggers

time-domain and frequency-domain
heart rate variability analysis

patient event markers, automatic cuff
size detection, voltage display
to verify battery status

CardioTens

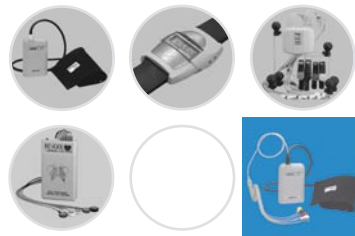

ambulatory blood pressure
and ECG monitor

CardioTens is an innovative ambulatory diagnostic tool that highlights the relationship between coronary heart disease and hypertension and provides a detailed analysis of various parameters of cardiovascular risk. It records 24-hour blood pressure, ST-segment and heart rate profiles, and two-channel ECG strips according to a preset program sequence. The unit performs continuous real-time beat-to-beat ECG analysis as well. Automatic event detection with adjustable preset limits ensures that extra ECG strips and corresponding blood pressure values are recorded during (silent) ischaemic events, high or low heart rates and syncopal episodes. Patients can mark and record additional, symptomatic events. CardioTens also performs full heart rate variability analysis. It uses the BHS and AAMI validated proprietary Meditech ABPM algorithm, as well as the Meditech ECG processing algorithms successfully tested on the MIT-BIH arrhythmia database (99.8 % beat detection accuracy). The device stores all recorded data in its solid-state memory even if batteries are removed. CardioTens offers on-line ECG monitoring with real time cycle averaging and ST detection to help achieve the best electrode placement and perfect adjustment of event detection triggers.

Technical parameters

power supply	4 high capacity AA rechargeable batteries
BP measurement	oscillometric method, stepwise deflation, piezo-resistive sensor
measurement range	blood pressure 30-260 mmHg, pulse rate 40-200 bpm
passive accuracy	± 3 mmHg or 2 % of measured value
ECG parameters	two channels, 200 Hz sampling, 12 bit A/D resolution
BP storage capacity	max. 1000 ABPM measurements
ECG storage capacity	24-hour ST, ST-slope, HR and HRV; at least 4 hours of 2-channel ECG recordings
data transfer	RS232 line, optical cable transmission, 115.200 baud
dimensions	124 x 82 x 33.5 mm
weight	~ 350 g (incl. batteries)



P.M.S (Instruments) Ltd
Waldeck House
Waldeck Road
Maidenhead
SL6 8BR
UK
Tel 01628 773233
Fax 01628 770562
email sales@pmsinstruments.co.uk

www.meditech.hu