PulmoLife

INSTRUCTIONS FOR USE

MODE D'EMPLOI

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INSTRUCTIONS FOR USE

English

Page3

MODE D'EMPLOI

Français

Page9

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Deutsch

Page15

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Español

Page21

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Nederlands

Page27

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Italiano

Page33

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Português

Page39

References

Références

Hinweise

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
Referenties

Bibliografia

Referências

Page46

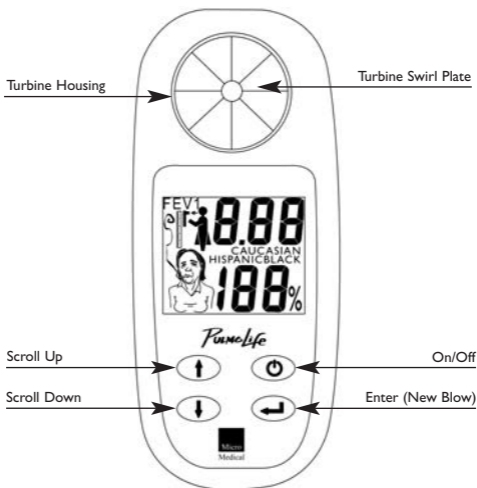ABOUT PULMOLIFE

PulmoLife quickly and easily screens smokers for early detection of Chronic Obstructive Pulmonary Disease (COPD).

The effects of COPD in smokers is normally detectable from the age of 35.

The unit measures the Forced Expiratory Volume in the First Second of Expiration (FEV_1) and the FEV_1 as a Percentage of the Predicted Value ($FEV_1\%$ predicted). The use of FEV_1 is a useful diagnostic tool for early detection of COPD¹. The results can also be used to calculate and display an equivalent 'Lung Age'²

This information can help in the diagnosis, management and treatment during the early stages of COPD development.

The use of 'Lung Age' provides an additional motivational tool that may help susceptible smokers to understand the physical damage caused by smoking.

BEFORE USING YOUR PULMOLIFE YOU MUST REMOVE THE BATTERY CONNECTION TAB

ENTERING PATIENT DETAILS

Patients predicted values are based on their Sex, Height, Age and Race (default predicted value set shown on box, to change see Predicted Value Set Up)

Switch the unit On  to begin.

When entering patient details use the Scroll Up  / Down  keys to change values and the Enter  key to accept and proceed.

Enter Sex (Male/Female)

Enter Height (cm for ECCS and Asian, inches for NHANES III)

Enter Age (Default is 45)

Enter Race (for ECCS and NHANES III only)

The display will show the blow symbol  and 0.00.

PERFORMING A TEST

Also see further instructions in the PulmoLife box.

Insert a new disposable one-way valve mouthpiece into the turbine holder.

With the display showing the blow symbol  and 0.00.

Instruct the patient to breathe in until their lungs are completely full, seal their lips around the mouthpiece, and blast out as hard and as fast as possible for at least one second. After one second the unit will emit a beep to signal the end of the blow.

IMPORTANT NOTE: *The rear of the turbine must not be obstructed during the blow.*

The FEV₁ and FEV₁% predicted will be displayed.

If a blow is too slow at the beginning of the manoeuvre³, the blow icon  will flash assessing this test as poor quality and the test should be repeated. After a full inspiration the patient should be encouraged to blast out the air as fast as possible without hesitation.

When FEV₁% predicted is less than or equal to 80% of predicted, the FEV₁% predicted value will flash indicating the possibility of airways obstruction.

Press the Scrolling Down  key to display the 'Lung Age' (range 20-90 years) and smoker visual. A patient's 'Lung Age' is limited to 30 years over their chronological age up to a maximum age of 90.

PLEASE NOTE: *If a patient's FEV₁% predicted is above 100% of predicted then their estimated 'Lung Age' would be lower than their chronological age.*

Use the scrolling Up  / Down  keys to move back and forth between both results screens.

Press Enter  to perform a new blow.

It is recommended that a spirometry result should be taken from the best of 3 technically correct blows.

To test another patient the unit must be switched off and on and then enter the new patient details.

CLEANING

Micro Medical Ltd recommends the use of One-Way Valve Disposable Mouthpieces for all multi-patient use devices.

To remove the turbine, simply press the turbine from the rear, without excessive force, and the turbine will pop out.

The turbine may be immersed in warm soapy water for routine cleaning.


To sterilise, immerse the turbine in a cold-water solution (e.g. PeraSafe) for a period not exceeding 10 minutes.

NOTE: *Do not use alcohol or chloride based solutions.*

After cleaning/sterilising, the turbine should be rinsed in distilled water and left to dry.

Do not use objects, such as pens or cotton buds, to clean or dry the turbine, as this may cause irreparable physical damage.

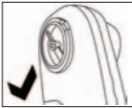To replace the turbine hold the unit and the turbine swirl plate facing you, put the turbine back into the turbine housing then push until the turbine is fully located, such that the back of the turbine is completely level with the back of the unit.

The PulmoLife moulding can be wiped clean using a damp cloth or surgical wipe.

CALIBRATION CHECK

The unit should not require recalibration. However, as a confirmation of the correct functioning of the unit, the calibration can be checked using a 3-litre calibration syringe.

To check the calibration hold down the On  and Scrolling Up  keys together, and only release when 'Ch' is displayed.

When the blow symbol  and '0.00' appear, empty the 3-litre syringe smoothly into the unit. The value will be displayed.

If the unit detects an error the blow symbol and calibration value will flash. An acceptable calibration check is within +/-3% of the 3-litres entered (so between 2.91 and 3.09 litres).

PLEASE NOTE: Calibration check failure is unusual but is most often due to malfunction of the turbine, either because it is physically damaged or dirty. Clean the turbine (see cleaning) and recheck the calibration. If the unit still fails calibration check, replace with a new turbine (see accessories for order number).

SERVICE

This unit requires no routine servicing.

BATTERY

If the battery level is low the unit will display the letters 'bAt'. If switched on with a low battery level, the unit will beep three times and then turn off after 5 seconds.

The battery is contained beneath the slide cover in the rear of the unit. To replace, simply slide the cover away, change the battery and replace the cover.

PLEASE NOTE: *If the battery is removed for any reason the unit will reset to factory predicted value settings.*

Replacement 3V coin cell batteries are available at most major electrical outlets.

Please be sure to dispose of the lithium ion coin cell battery according to standard local environmental procedures.

PREDICTED VALUE SET UP

To change the Predicted Value Settings, from the default shown on the box, hold down the On  and Enter  keys together, and only release when 'Prd' is displayed. Use the Scrolling Up  / Down  keys to choose an alternative Predicted Value set.

1 ECCS⁴

2 NHANES III⁵

3 Asian (Chinese)⁶

When the desired number is selected press the Enter  key to accept and save that predicted value set as default.

VERSION

To check the version number, hold down the On  and Scrolling Down  keys together, and only release when 'rE' is displayed. The version number will then be shown.

ACCESSORIES

Adult disposable mouthpieces with one-way valve (Box of 500)	SST1000
PeraSafe Sterilizing Solution 8 l g	SSC5000
Pouch	MIM-088-22
Replacement Turbine	36-ASS1148

SPECIFICATIONS

Transducer:	Digital Volume Transducer
Resolution:	0.01L
Accuracy:	+/-3% To ATS/ERS Taskforce Standardisation of Spirometry, 2005
Volume Range:	0 - 8 L As per ATS/ERS recommendations
Flow Range:	0-14 L/s As per ATS/ERS recommendations
Predicted Values	1) ECCS 2) NHANES III 3) Asian (Chinese)
Display:	Custom Liquid Crystal
Power Supply:	3V Lithium Ion Coin Cell Battery (model no CR-2450)
Dimensions:	131 x 59 x 38 mm
Weight:	Unit only: 96g Unit complete with accessories: 260g
Operating Temperature:	0° to +40° C
Operating Humidity:	30% to 90% RH
Storage Temperature:	-20° to + 70° C
Storage Humidity:	10% to 90% RH

SYMBOLS


Type B device

**In accordance with
Directive 93/42/EEC**

ENVIRONMENT

This instrument complies with directive EN60601-1-2 electromagnetic compatibility but can be affected by cellular phones and by electromagnetic interference exceeding levels specified in EN 50082-1:1992

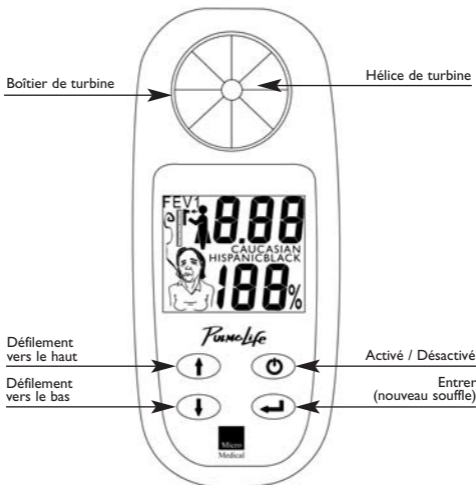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Micro Medical Limited