

BLOOD
PRESSURE
CUFF BARRIERS



Single patient use blood pressure cuff barriers.
Soft FABRICEL tissue fused with leak proof polyethylene film
Available in adult, large adult or paediatric sizes
Will not alter readings, latex free
Adult (300 covers) £59
Large adult (150 covers) £49
Paediatric (300 covers) £59

Prices exclude delivery and VAT.



Prices exclude carriage and VAT.

Quick and easy to use:



- 1 Remove one Barrier from dispenser box.



- 2 Remove strip to expose adhesive.



- 3 Place tape end of Barrier on top of patient's arm, poly out, tissue against patient's skin.



- 4 Holding tape end in place, wrap patient's arm and press Barrier's free end against adhesive.



- 5 Apply blood pressure cuff directly over Barrier.

SAFETY

CONVENIENCE


- Applies in seconds
- Will not alter readings
- Reduces cleanup of blood pressure cuff

COMFORT

- Soft tissue against skin
- Latex-free
- For use on arms or legs



Hospital

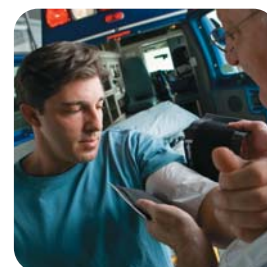

Emergency

PMS

P.M.S. (Instruments) Ltd

P.M.S (Instruments) Ltd
Waldeck House
Waldeck Road
Maidenhead
SL6 8BR

Tel 01628 773233

Fax 01628 770562

email

sales@pmsinstruments.co.uk