

# DIGITAL BLOOD PRESSURE MONITOR

## UA-779

### Fuzzy Logic



### *Fully Automatic One-Button Operation with Memory*

- Easy one-button operation.
- Fuzzy logic determines ideal cuff inflation level.
- Easy-to-read display shows systolic, diastolic and heart rate simultaneously.
- Automatically stores last reading in memory.
- Oscillometric method ensures hospital accuracy for everyday use.
- Multi-lingual instruction manual.



**A&D** ...Clearly a Better Value  
A&D Company, Limited

# DIGITAL BLOOD PRESSURE MONITOR UA-779

## Fuzzy Logic

### Easy one-button operation

A single button activates the following functions: power on, start, cuff inflation, blood pressure measurement, data display, measurement storage in memory, cuff deflation and power off.

### Fuzzy logic determines ideal cuff inflation level

Software technology built into the UA-779 automatically senses the correct inflation level, about 30 mmHg over the estimated systolic pressure, eliminating the need for a preset switch.

### Easy-to-read, large character display

After each measurement, the UA-779 simultaneously displays systolic and diastolic pressure and heart rate on large, clear 16mm character display.

### Automatically stores last reading in memory

After each measurement, the UA-779 automatically stores the reading in memory. To recall the stored reading, just press the memory button at anytime.

### Oscillometric method ensures hospital accuracy for everyday use


The UA-779 utilizes the same Oscillometric technique used by hospital bedside monitors. With proprietary A&D software, the UA-779 assures reliable and accurate blood pressure measurement at home.

### Multi-lingual instruction manual

The UA-779 operating manual includes instructions in five languages: English, Spanish, French, Italian and Chinese (Mandarin).

### World Health Organization Blood Pressure Standards

At present the WHO(World Health Organization)has set up four classifications of blood pressure which are not related to the subject's age. They are outlined in the chart.

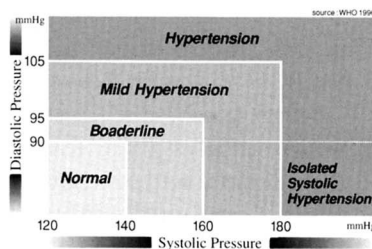

**A&D**  
A&D Company, Limited  
Tokyo JAPAN

REF \* CARD-UA779-990718-02PR1